



# Practical innovation

Pico nasal mask delivers performance for you and your patients

**PHILIPS**  
RESPIRONICS

At Philips Respironics, we created Pico with a definitive goal in mind — to create a truly appealing traditional nasal mask that makes minimal contact with the face and offers simplicity, comfort and seal. These qualities make it an ideal choice for new and existing sleep apnea patients who want minimal disruptions to their lifestyle.

**Pico is the lightest and smallest traditional nasal mask on the market.\***

Pico is small, light, and comfortable – while providing a high level of therapy.\*

**Pico comfortably fits more than 98% of patients\***

Pico offers patients a simple-to-use mask with an excellent seal that offers the freedom to watch television and read while wearing it.

Clinicians rated Pico easier to use, setup, and obtain a seal than the leading traditional nasal masks, making it ideal for your installed base optimization and new patient set-ups.\*

\*Data on file. Internal Marketing Survey 2013.

# Get started

To learn more about how Pico can start making a difference for your patients and your business, contact your Philips Respironics representative for more information.



Item	Part number	HCPCS Code
Small/medium, large, extra large Pico nasal mask Fitpack, with headgear	1104940	A7034,A7035
Small/medium Pico nasal mask, with headgear	1104915	A7034,A7035
Large Pico nasal mask, with headgear	1104916	A7034,A7035
Extra large Pico nasal mask, with headgear	1104917	A7034,A7035
Small/medium Pico nasal mask, without headgear	1104918	A7034
Large Pico nasal mask, without headgear	1104919	A7034
Extra large Pico nasal mask, without headgear	1104920	A7034
Replacement parts	Part number	HCPCS Code
Pico headgear	1104934	A7035
Pico headgear, reduced size	1104935	A7035
Small/medium nasal cushion	1104936	A7032
Large nasal cushion	1104937	A7032
Extra large nasal cushion	1104938	A7032
Pico sizing gauge	1104939	N/A
Pico headgear clips	1104933	N/A
Pico quick release swivel	1104931	N/A

## The ANSWER

Transform your business the easier way


Pico is the latest in our comprehensive portfolio of advanced mask designs plus powerful resupply solutions and innovative technologies. It's all designed to work together to your distinct advantage at a time when you need it most.





Philips Respironics is working in harmony with the care team to produce solutions that can lead to healthier patients, healthier practices and healthier businesses.


# Pico fitting guide


**Achieving the right fit with Pico is easy; just follow the instructions below to get started. See OIS fitting guide for complete instructions.**


- 


1 Before putting on the mask, make sure the cushion fits the width of your nose without blocking your nostrils. The top of the cushion may sit between the bridge and tip of your nose.
- 

2 Disconnect the headgear clips by twisting them away from the frame like a key and undo the headgear tab by peeling it away from the headgear strap.
- 

3 Pull the headgear on over your head and gently place the cushion over your nose.
- 

4 Now, while holding the mask in place, connect the headgear clips by pressing onto the mask frame.
- 

5 Adjust the top and bottom straps by pulling the tabs back evenly. The mask should rest lightly on your nose. Do not over-tighten. The headgear should fit loose and comfortable.
- 

6 Next, connect Pico to the flexible tubing that came with your sleep therapy device. Some air leaking is normal. Adjust the side headgear straps to reduce leaks around your eyes.
- 

7 Assume different sleeping positions. Move around until comfortable. If there are any excessive air leaks, make final adjustments while lying down.

**Philips Healthcare is part of  
Royal Philips**

**How to reach us**

[www.philips.com/healthcare](http://www.philips.com/healthcare)  
[healthcare@philips.com](mailto:healthcare@philips.com)

**Asia**

+49 7031 463 2254

**Europe, Middle East, Africa**

+49 7031 463 2254

**Latin America**

+55 11 2125 0744

**North America**

+1 425 487 7000

800 285 5585 (toll free, US only)

Philips Respiroics  
1010 Murry Ridge Lane  
Murrysville, PA 15668

Customer Service  
+1 724 387 4000  
800 345 6443 (toll free, US only)

Philips Respiroics Asia Pacific  
+65 6882 5282

Philips Respiroics Australia  
+61 (2) 9947 0440  
1300 766 488 (toll free, Australia only)

Philips Respiroics Benelux  
+33 1 47 28 30 83

Philips Respiroics China  
+86 400 828 6665  
+86 800 828 6665

Philips Respiroics Deutschland  
+49 8152 93 06 0

Philips Respiroics France  
+33 2 51 89 36 00

Philips Respiroics Iberia  
+34 91 566 90 00

Philips Respiroics Italy  
+39 039 203 1

Philips Respiroics Sweden  
+46 8 120 45 900

Philips Respiroics Switzerland  
+41 6 27 45 17 50

Philips Respiroics United Kingdom  
+44 800 1300 845

[www.philips.com/respiroics](http://www.philips.com/respiroics)

Respiroics and Pico are trademarks of Respiroics, Inc. and its affiliates.  
All rights reserved.

Please visit [www.philips.us/pico](http://www.philips.us/pico)



© 2015 Koninklijke Philips N.V. All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from use of this publication.  
CAUTION: U.S. federal law restricts these devices to sale by or on the order of a physician.

RRDPGH MG 01/08/2015 MCI 4106522 PN 1120553